


# RC High Efficiency Leucocyte Removal Filter

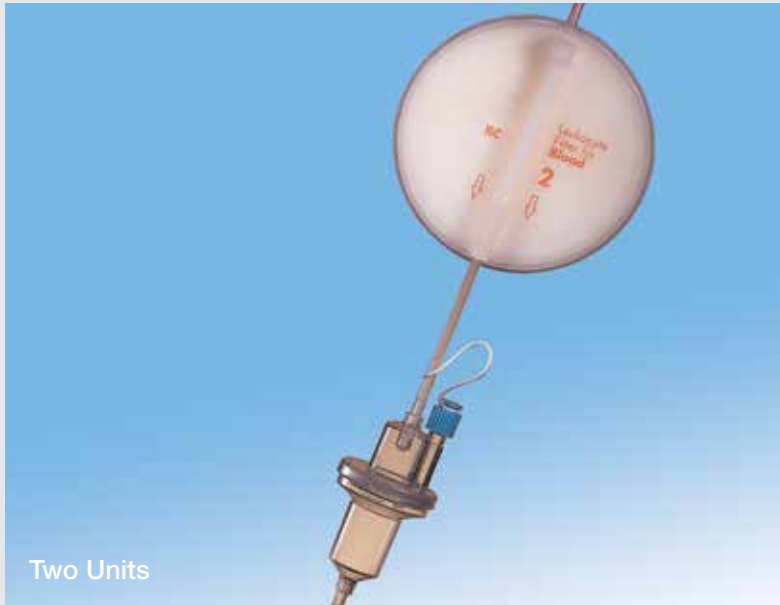

## For Blood Transfusion

- Clinically proven media technology
- For standard or rapid flow applications
- Easy prime technology
- Enhanced ease of use
- High efficiency leucocyte depletion
- High red cell recovery
- Minimal filter hold-up volume
- Rapid priming without saline
- Bedside filtration of two units of red cells

## Features and Benefits

- Clinically proven media technology significantly reduces the risk of leucocyte associated transfusion complications such as microaggregates, alloimmunisation, febrile reactions, refractoriness to platelets, Cytomegalovirus and immunosuppression.\*
- Ease of use has been significantly enhanced by the new filter design which provides:
  - Priming by gravity or rapid priming by squeezing the blood bag
  - Unique self levelling drip chamber allowing self priming of the filter and drip chamber
  - New vented spike design to allow upstream of filter to drain following transfusion, maximising red cell recovery
- Dependably and efficiently delivers low leucocyte residuals, affording the maximum patient protection against leucocyte related transfusion complications (consistently averaging less than  $2 \times 10^5$ /unit for buffy coat poor red cells).\*
- Primes directly with red cells quickly and conveniently without the need for priming with saline.
- High technology filtration media and minimal filter hold-up volume (31 mL after recovery) provides minimal loss of red cells without the need for saline flushing.
- Unique housing design allows maximum use of the filter media surface area for consistent results.



Vented Spike



Self Levelling Drip Chamber

\* Data available on request from Haemonetics Corporation.

## Performance Summary

The residual WBC level after filtration of 2 units of buffy coat depleted red cells through an RC2 filter at either gravity flow or high flow consistently averages less than  $2 \times 10^5$ /unit.

| PRC Additive | Blood Age (Days) | Pre-Filtration<br>(x 10 <sup>9</sup> /unit)** | Post-Filtration<br>WBC (x 10 <sup>5</sup> /unit)** | Flow Rate |
|--------------|------------------|---|--|-----------|
| SGM-BC       | 3                | 0.28  | 1.00   | 1         |
|              | 3                | 1.11  | 1.26   | 2         |
|              | 4                | 0.98  | 1.18   | 1         |
|              | 4                | 0.56  | 1.03   | 2         |
|              | 7                | 2.59  | 0.26   | 2         |
|              | 16               | 0.23  | 0.46   | 1         |
|              | 30               | 0.83  | 0.29   | 1         |
|              | 30               | 0.55  | 0.27   | 2         |
|              | 31               | 1.06  | 0.60   | 1         |
|              | 31               | 0.63  | 0.43   | 2         |

\*\* = Average of unit 1 + unit 2

1 = 1m gravity flow

2 = 300 mmHg pressure

- WBC count post filtration was determined using a manual counting method (Nageotte Chamber).
- Blood was stored at 4 °C and left for 10 minutes at room temperature before filtration.

## Ordering Information

| Reorder Code | Description   | Pkg     |
|--------------|---|---------|
| RC2VE        | RC High Efficiency Leucocyte Removal Filter                 | 20/case |
| RC2VAE       | with Attached Self Levelling Administration Set             | 20/case |
| RC2VAYE      | with Y-inlet and Attached Self Levelling Administration Set | 20/case |

Information contained herein for distribution outside of the USA only.

© 2002, 2013 Haemonetics Corporation. Haemonetics and Haemonetics The Blood Management Company are trademarks or registered trademarks of Haemonetics Corporation in the USA, other countries, or both. All rights reserved. 04.2013 USA. COL-COPY-000572-IE(AA)